



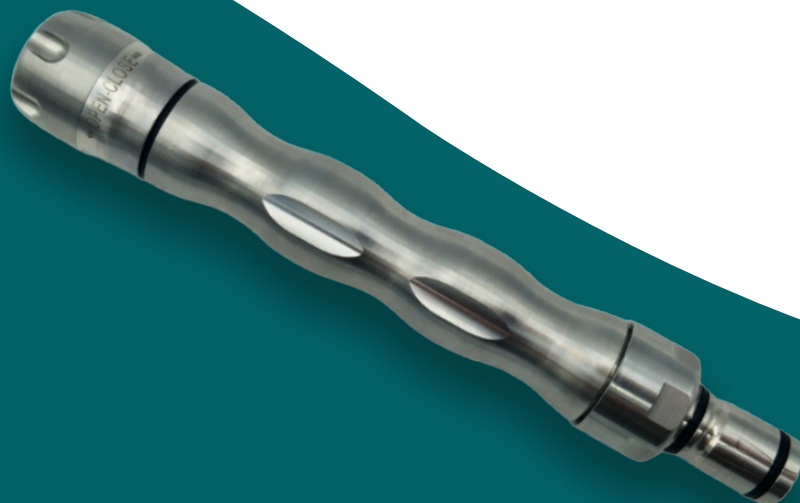

USER'S NOTICE


Easy

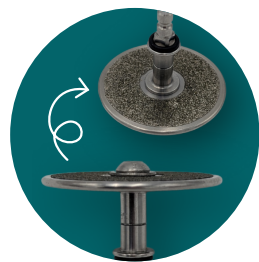

Diamond insert connection

Caution!

The screw direction is reversed from the standard direction; follow the markings on the handle to guide you.

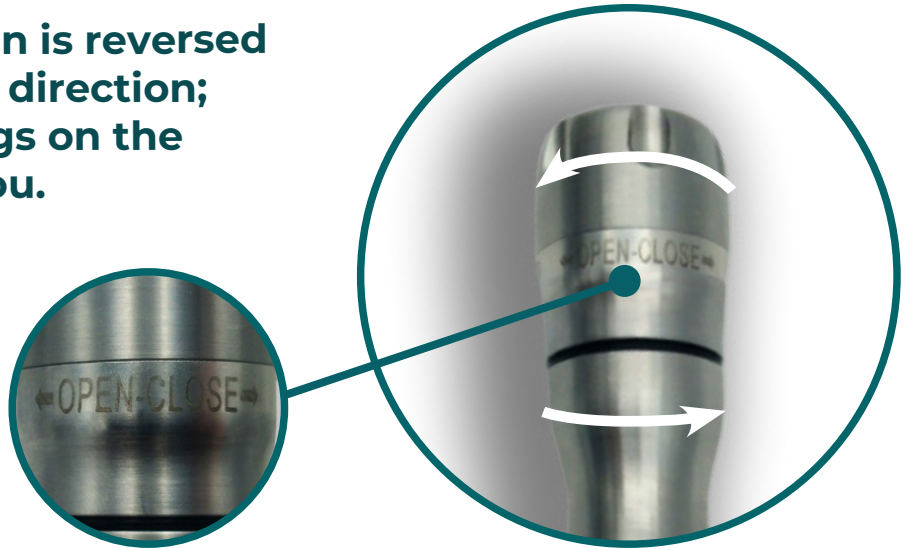

Procedure for replacing diamond inserts



Unscrew the head and remove the diamond insert



Insert the new diamond insert, making sure to align the square



Screw it back on tightly, and your Easy burr is ready to use

Maintenance Recommendations

1 General recommendations

Make sure the diamond insert is securely attached before you begin working.

Before storing your equipment, make sure it is clean and dry.

It is important not to leave any water in the connectors, as this could damage the equipment.

Always store the equipment with a diamond insert in place.

Vet-Design recommends annual preventive maintenance.

2 Diamond inserts maintenance

Évitez de laisser vos fraises immergées dans l'eau.

Pour le nettoyage de la fraise, Vet-Design recommande de mettre en marche le moteur.

- Brossez la tête diamantée en fonctionnement à l'aide d'une brosse inox pour éliminer les dépôts de dentine.
- Rincez et/ou désinfectez, puis assurez-vous de bien évacuer le liquide en faisant tourner la fraise hors de l'eau.
- Bien sécher à l'aide d'un chiffon doux, propre et sec.

Adress

Vet-Design
7 Ter Rue du Clos Michel
35 590 Saint-Gilles
FRANCE

Contact

+33(0)2.21.81.11.71
contact@vet-design.com

